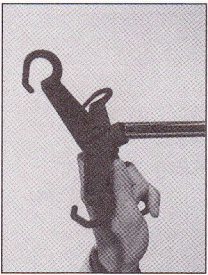
3.2 Montage des Flaschenkreuzes:

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3.2 Assembling the bottle holder:

- Put the bottle hanger onto the ø18 tripod tube.

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4. Cleaning and Disinfection:

The infusion stand can be treated with mild cleaning agents, for the disinfection you should use commercially available disinfection agents for example: Amocid, Baktolan, Sagrotan.

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5. Maintenance and Repair:

The infusion stand should be checked at least once a year for proper functionality of all important components:

Clamp-function for the inner tube Ø18 still dequate. To assure functionality of the locking and antistatic rollers clean them up.

Repairs have to be done by authorized personal or customer service with original equipment only.

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6 Ambient conditions:

For transport and stock-keeping up to 3 months, the following stock-keeping conditions have to be heeded:

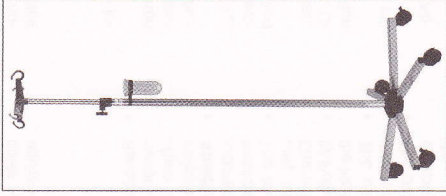
Temperature: -25 bis 70°C
Moisture: 30-75%
atmospheric pressure: 700-1060 hPa

At application you have to assure the ambient temperature from 10°C to 75°C.

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1. General / purpose:

- The IV-Pole series 411... with special screwed height adjustment is an IV-Pole for gravity infusions.
- To ensure that the comprehensive safety demands on spaces used for medical purposes are met to your satisfaction, the safety instructions have to be heeded.

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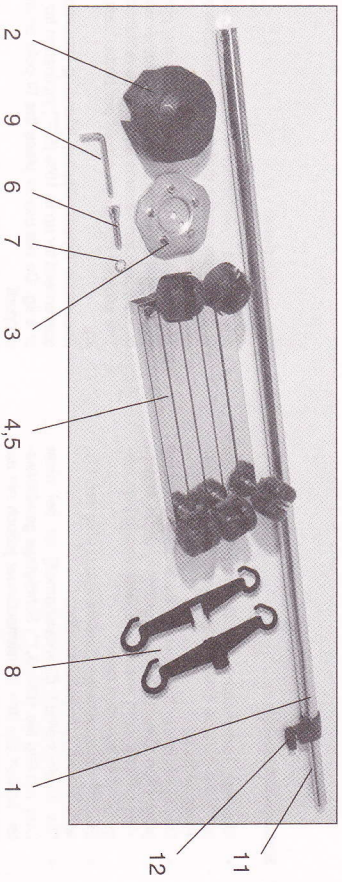
2. Safety instructions:

- It is absolutely necessary when setting up, handling and using the device that these instructions are followed precisely.
- The devices are only to be used for the purposes named in the directions for use. Any other use may conceal unknown hazards and is to be refrained from.
- The allowed maximum weight per bottle-hook is 2,0 kg.
- Assurance against tipping (standing assurance) is guaranteed for up to 10% (5,7°) inclination for a load of 8 kg. Do not use or place the IV-pole on inclined support!
- Heavy objects like lead aprons etc. are not to be hung on the IV-Pole.
- When moving, make sure that the roller-brakes have been unlocked. The IV-Pole has to be moved by hand only! Rolling across obstacles, soft flooring and slants must be done with particular care. If the slope is more than 10%, then the inner tube has to be at shortest / min. position.

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3. Assembling

- Pumps and other devices must only be mounted if the steadiness is always granted. We recommend to place the devices as low as possible only to the outer tube and use a maximum weight for extra equipment of:
 - 1,5 kg for IV-Poles without extra weighted legs and
 - 3,0 kg for IV-Poles with extra weighted legs.
- Because extra equipment can be adapted in numerous variations, it is in every case necessary to check if the steadiness is still warranted before commissioning.
- **Attention:** Do not open the screw for height adjustment (12) without holding the inner tube (11) because the load might slide down uncontrolled.
- The height adjustment screw has to be tightened only as much as the load is hold tight secure. Overwinding the screw might damage the IV-Pole.
- IV-Poles not in use are always to be locked with the lock rollers.
- The electrically conductive rollers serve only to divert static charges.
- The person who is selling the product, is only responsible for the safety and proper functioning of the IV-Poles when the directions for assembly and use have been followed in all points and only when original accessories are used.

Box contents:

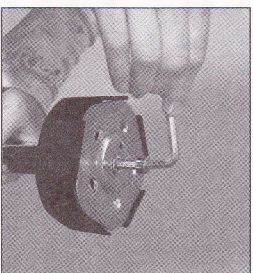
- 1x IV-Pole
- 1x plastic mid-section
- 1x clamp disk with grooves
- 2x feet with locking rollers (antistatic)
- 3x feet with roller
- 1x hexagon socket head cap screw M8
- 1x toothed disk
- 2x plastic bottle-holder half's
- 1x Hexagon key
- 1x Directions for assembly and use

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
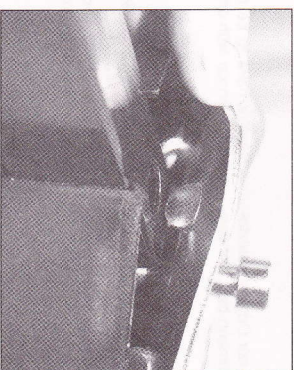
3

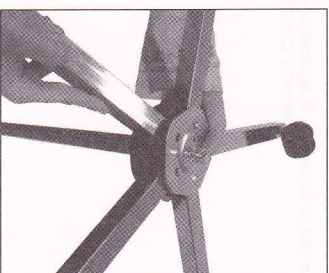4

- **3.1 Assembling the tripod foot:**
- Slip the plastic mid-section (1) onto the Ø25 tube (2). The tightness enables a movement-free seating.

- Pre-assemble the clamp disk with grooves (facing inward) with the toothed disk and hexagon socket head cap screw so that the feet can still be easily inserted in the footed mid-section.

- Insert the first leg a bit angled into the footed mid-section. Be sure that the feet is inserted up to the back-stop, cause the base must be holded by the grooves of the clamp disk. (grooves must snap into the holes of the leg)

- Now mount the last four feet. It is important that all feet are fully inserted in the footed mid-section, to make sure that all grooves are fixed in the holes. Please consider that - to have the right mounting order - you first need to mount the opposite feet and that the feet with brakes are not placed next to each other.

- Now, with an Hexagon key, tighten the clamp disk.